



www.healthcraftproducts.com

Invisia™ Bath Board

Installation Instructions: WS-BB-PCG

THANK YOU...

Thank you for purchasing the HealthCraft™ Bath Board. We are confident that you will find the durable construction of this product to be of great assistance in your daily routine.

To fully enjoy your Bath Board, please take a few moments to read these instructions. You should store these instructions in a safe place for future reference.

SAFETY CONSIDERATIONS

Bath Board can be slippery when wet - use this product with caution. Product intended to be used as a sitting surface only. Do not stand on the bathboard. Do not tip or put an offset load onto the bathboard or injury may occur. If board overhangs past the tub rim, the area of the board outside of the tub should not be used for seating. Inspect frequently to ensure all screws are tight.

WOOD PANEL WARNING

In order to retain the natural beauty of the Brazilian Walnut (IPE), each slat is finished with an extremely durable 2-part oil finish that does not contain volatile organic compounds (VOCs) or other harmful chemicals.

A light brown or red discoloration may be noticed in water exposed to the wood, and is due to an excess of pigments (natural oils) in the wood reacting with minerals present in certain types of water (well water, hard water, etc.). This discoloration is a natural reaction often referred to as leaching and should fade over time. The applied oil finish also reduces the effects of leaching. If unsure, test wood in local water prior to installation.

WARRANTY

HealthCraft's Bath Board has a limited lifetime warranty with the exception of parts which are subject to normal wear. Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft™ Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect.

HealthCraft™ Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft™ Products Inc.


HealthCraft™ Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft™ Products Inc. prior approval. HealthCraft™ Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

GENERAL DIMENSIONS


	DESCRIPTION	DISTANCE
A	Length of board	29.5" / 74.9cm
B	Width of board	11" / 28cm
C	Maximum distance between friction feet	27.25" / 69.2cm
D	Minimum distance between friction feet	19.75" / 50.2cm

INSTALLATION INSTRUCTIONS

1. Ensure there is a minimum horizontal flat surface of 1.25"/32mm (X) on the tub rim for the bath board overhang to rest upon as shown below. If factory-installed bumpers do not line up with tub rim, see **Bumper Install** below. Adjust friction feet to fit snug inside bathtub (a). Thread friction feet onto the plastic screw to secure in place (b).

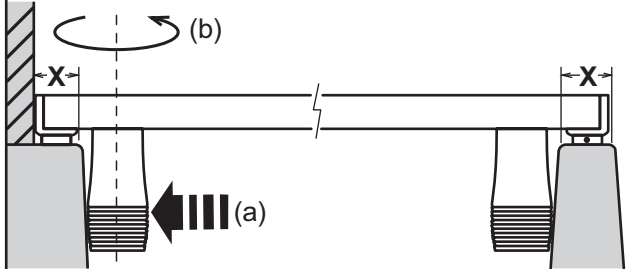

BUMPER INSTALL. If required, install the extra bumpers directly to the wood. Carefully drill a pilot hole for the (supplied) #6 pan head wood screws. Ensure the bumpers are aligned with factory-installed bumpers as shown in Bottom View.

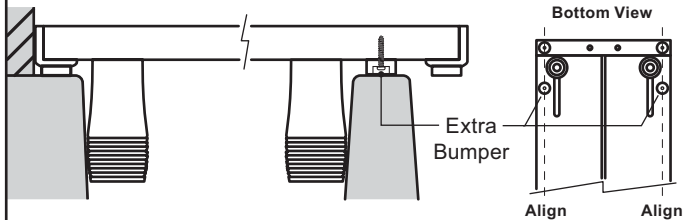

PARTS DIAGRAM

