To assemble:

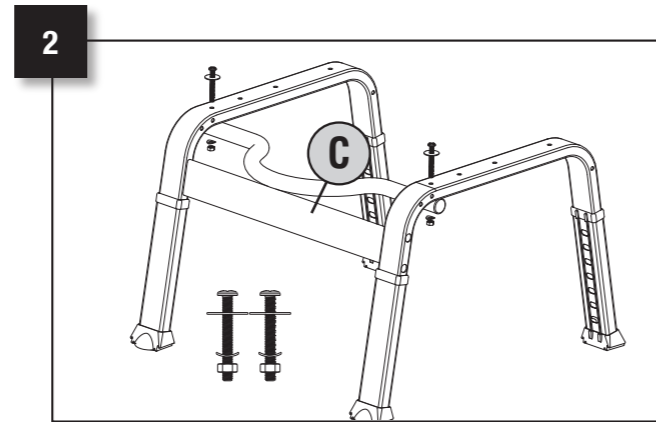
1. Position cross support (A) between legs (B). Use four 2" screws (found in the cross support screw package) in holes located on legs and tighten. Once all screws are in place, insert plastic caps to cover them.

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
2. Attach seat support bar (C) to legs using two 2 1/4" screws, two flat washers, two curved washers and two nuts provided in screw package (I).

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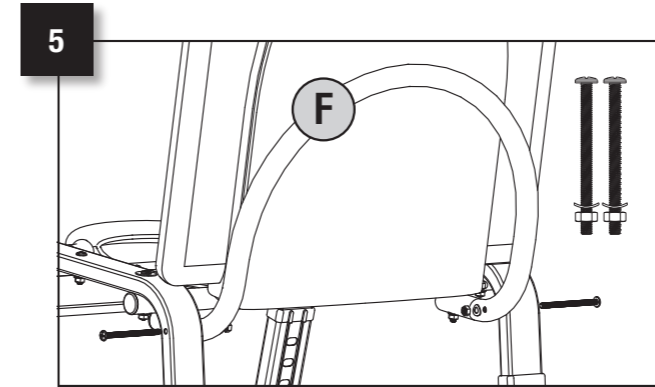
5. Attach back support (F) to legs using two 3/4" screws, two curved washers and two nuts from screw package (I). Align side holes in back support with the appropriate holes in the legs. Insert screws, washers and nuts, then tighten.

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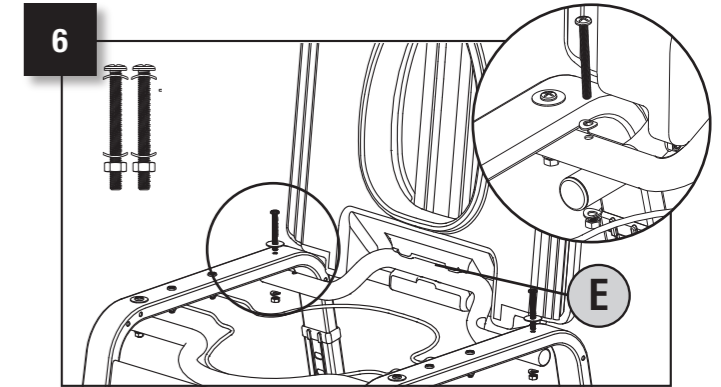
6. Complete installation of back support using two 2 1/2" screws, four curved washers and two nuts from screw package (I). Insert screws, washers and nuts through the inner holes in metal seat assembly bar (E) and top holes in back support, then tighten.

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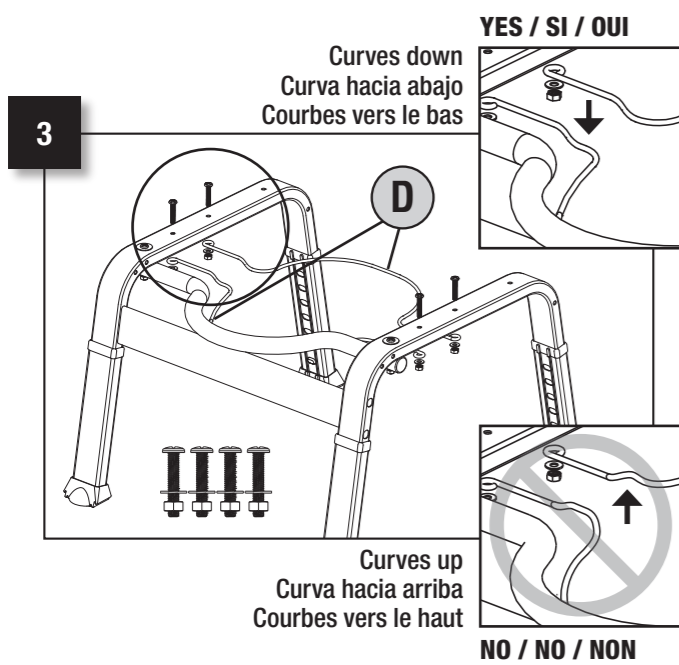
3. Attach the front and rear Bucket support wires (D) to the legs with four 1 1/4" screws, four flat washers and four nuts (found in support wire screw package).

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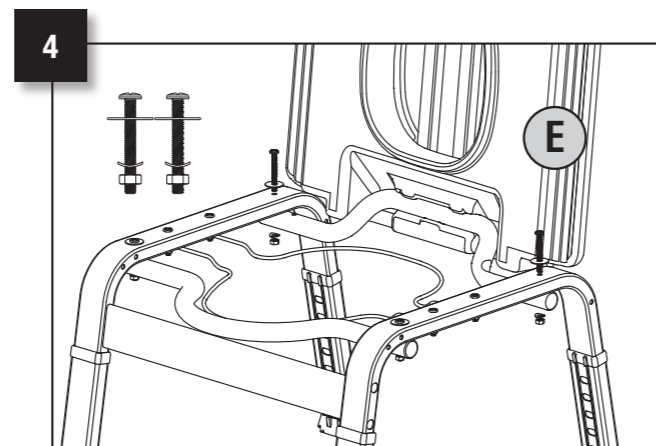
4. Install the seat assembly (E) by using two 2 1/4" screws, two flat washers, two curved washers and two nuts from screw package (I). Align outer holes in metal seat assembly bar under the appropriate holes in the legs. Insert screws, washers and nuts, then tighten.

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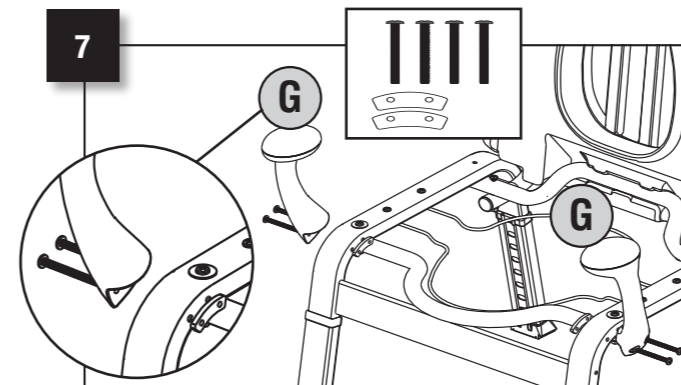
7. Attach right and left handles (G) to legs using four 2 1/2" screws, two threaded plates (found in handle screw package). Threaded plates should be placed on inside surface of leg.

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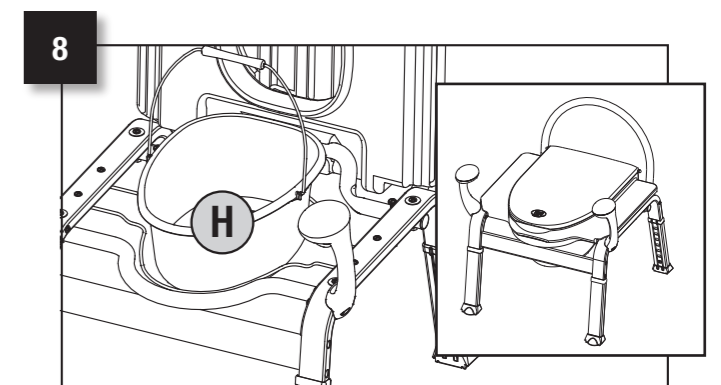
8. Lift seat assembly to insert bucket (H) with handle toward the back. Close lid.

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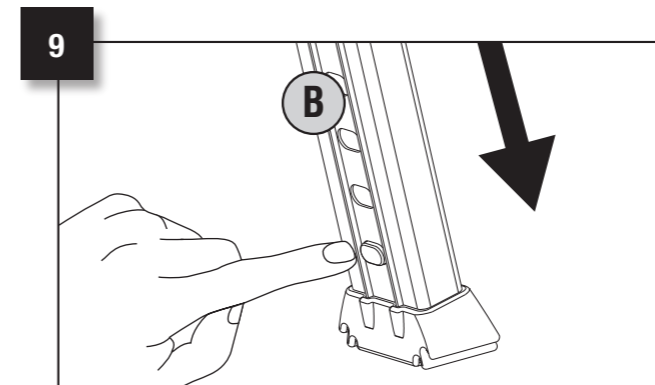9. Adjust bedside commode to the appropriate height. To do so, push adjustment buttons located on inside of all four legs (B) and slide leg until it reaches the desired height.

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INNOVATIVE DESIGNS
for independent living™

WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 400 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-289-6636.

LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn.

TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.homecare.moen.com

Home Care by Moen Premium Shower Chair meets or exceeds the applicable requirements of the following standards/guidelines:

- The Americans with Disabilities Act (ADA)

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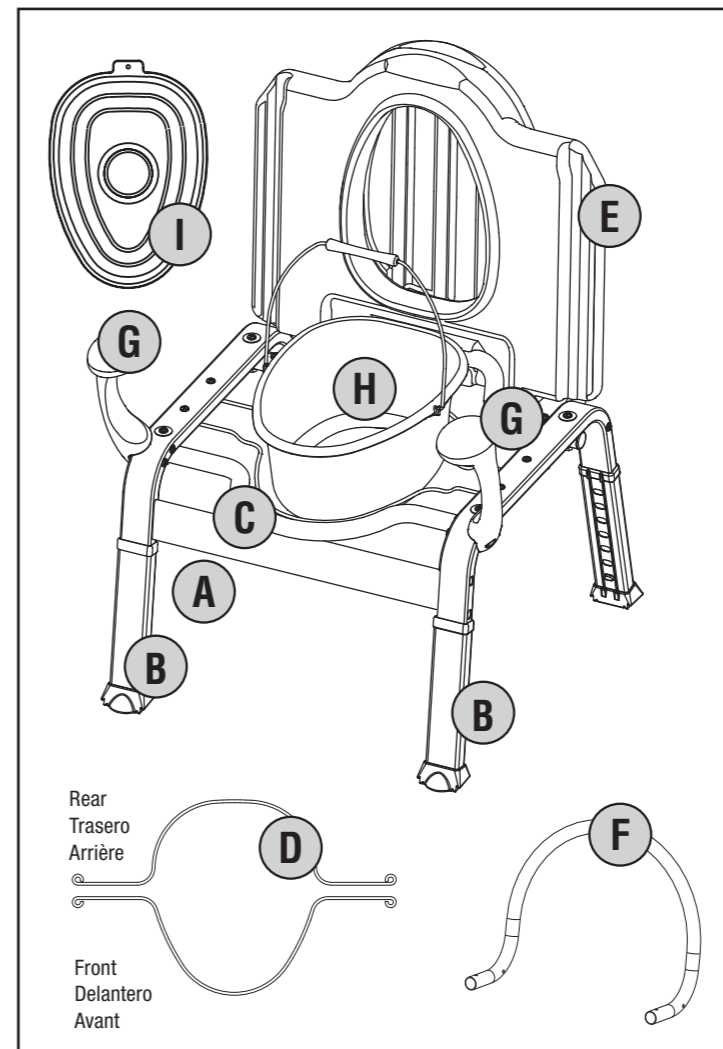
WARNING: PLEASE FOLLOW THESE ASSEMBLY INSTRUCTIONS CAREFULLY. ASSEMBLER ASSUMES RESPONSIBILITY FOR PROPER ASSEMBLY. FAILURE TO FOLLOW CORRECT ASSEMBLY PROCEDURES CAN RESULT IN SERIOUS INJURY. FOR MAXIMUM STABILITY, ALL FOUR CHAIR LEGS MUST BE ADJUSTED TO THE SAME LENGTH.

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Parts List

- A** Cross support with cross support screw package (four 2" screws; four plastic caps)
- B** Legs (right and left)
- C** Seat support bar (two 2 1/4" screws; two flat washers; two curved washers; two nuts)
- D** Bucket support wires (front and rear) with support wire screw package (four 1 1/4" screws; four flat washers; four nuts)
- E** Seat assembly (two 2 1/4" screws; two flat washers; two curved washers; two nuts)
- F** Back support (two 3 1/4" screws; two 2 1/2" screws; four nuts; six curved washer)
- G** Handles (right and left) and handle screw package (four 2 1/2" screws; two screw supports)
- H** Bucket
- I** Bucket Lid

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Estimated assembly time: 20 to 30 minutes

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Tools Required Herramienta requerida Outils requis

A phillips head screwdriver and adjustable wrench are required to assemble this product

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Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.

Home Care and Creative Specialties are a registered trademarks of Moen Incorporated.

Creative Specialties is a division of Moen Incorporated. © 2008

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